




iCup[®] Drug Test

Drug Test Interpretation Quiz

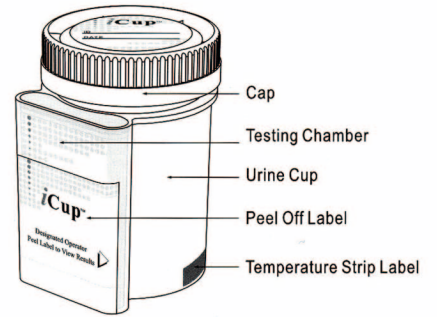

I. MULTIPLE CHOICE

- When collecting specimen in a cup, what should the temperature range be?
 - 90-100° Celsius
 - 85-95° Fahrenheit
 - 90-100° Fahrenheit
 - 85-95° Celsius

- The absence of a line next to a T or number line indicates a _____ for that particular drug.
 - Non negative
 - Negative
 - Invalid
 - All of the above

ANSWER: _____

- The specimen validity test or SVT strip checks for:
 - Oxidants
 - Specific gravity
 - Ph
 - All of the above

ANSWER: _____

- Negative results may be interpreted as soon as all lines are visible. Non negative results should only be interpreted at _____.
 - 10 minutes
 - 1 minute
 - 3 minutes
 - 5 minutes

ANSWER: _____

II. MATCHING (Match the appropriate abbreviation to the drug name)

- | | | |
|---------------------|-------|------|
| 1. Marijuana | _____ | COC |
| 2. Amphetamines | _____ | PCP |
| 3. Cocaine | _____ | mAMP |
| 4. Opiates/Morphine | _____ | AMP |
| 5. Phencyclidine | _____ | OPI |
| 6. Methamphetamine | _____ | THC |

Please fill out your contact information to receive a Certificate of Completion:

Name _____
 Company _____
 Address _____
 Phone (____) _____

III. IDENTIFY THE RESULTS AS EITHER: NON NEGATIVE, NEGATIVE, OR INVALID

